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The function test of your sealing device should be done according to the manufacturer's data. This check is done with a sterilisation role of at least 100 mm width. The whole process should be conform to your standardised practice processes and the data should be recorded on the control strip. By means of the Euroseal®-Check from EURONDA you verify the quality of your sealing seam and document the result.

#### Application of the control strip:

1. Switch the sealing device on and wait until ready-to-operate is signalled.
2. Place the Euroseal®-Check with the test field visibly into the sterilisation foil.
3. Then seal the foil exactly on the test field. The sealing seam should run centrally along the complete length of the test field.

#### Interpretation of the control strip:

1. Visual analysis of the sealing seam. The surface of the sealing seam has to be consistently visible with clear outlines along the complete test field. No inconsistent, perforated or torn parts are allowed.
2. For control purposes measure the sealing seam. It should be at least 8 mm wide everywhere. The sealing seams of the Euroseal® Valida, The Euroseal® and Euroseal® 2001 Plus should have a width of 12 mm.
3. Please record and sign the result in a separate documentation. It is advised to keep the latest check.

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