**DENOMINAZIONE / PRODUCT
NAME / NOM DU PRODUIT /
PRODUKTBEZEICHNUNG /
DENOMINACIÓN**

**GUANTI ANTITAGLIO
CUT PROOF GLOVES
GANTS ANTI-COUPURES
SCHUTZHANDSCHUHE
GUANTES ANTI CORTES
LUVA ANTI CORTE**

**DESCRIZIONE / PRODUCT
DESCRIPTION / DESCRIPTION /
PRODUKTBESCHREIBUNG /
DESCRIPCIÓN**

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Re-usable nitrile cut proof glove with an inner lining and an anatomical shape (right and left hand).

Outer anti-slip palm and finger finishes.

Use: indicated for use in sterilisation environments and during disinfection procedures in operating theatres.

Nitrile is latex-free and can therefore be used by latex-sensitive individuals. This material provides resistance to chemical agents, punctures and tears in order to increase protection from injury caused by contact with instruments.

The glove is especially long so as to prevent fluids and disinfectants from penetrating inside. This same feature also provides protection for operators when using chemical agents.

Suitable for autoclave sterilisation up to 121°C/249.80°F up to 5 times.

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CHEMICAL PRODUCTS	PENETRATION TIME (in minutes) ²	PERMEATION INDEX (1 to 6) ²
Methanol	40	2
n-heptane	>480	6
Sodium Hydroxide 40%	>480	6
Sulphuric Acid 96%	106	3

¹ **Mechanical risk:** The numbers refer to, in order, the levels of resistance to Abrasion (0 to 4), Cut (0 to 5), Tear (0 to 4) and Puncture (0 to 4). The higher the index number, the more resistant the glove is to the indicated challenge.

² **Penetration Time and Permeation Index:** Indicates the time needed for the chemical to penetrate into the glove. The higher the index number, the greater the permeation index.

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