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Irrigation handle, used together with a pressured bag or machinery irrigation system and used with dispensing liquid coolants in oral surgery. Non toxic. Apyrogenic. Free of latex components and phthalates.

Instructions for use:

1. Place the bag with dispensing liquid coolants into the cuff.
2. Open sterile peel pack with the irrigation handle into the sterile operation area.
3. Insert the spike into the bag with dispensing liquid coolants.
4. Pressurize the bag with the manual hand pump to desired level for the specific function. Recommended range is between 100 to 250 mmHg.
5. With manual pressurization, periodic monitoring and adjustment of pressure level is recommended during irrigation.
6. Flow is obtained by pushing down the button. Flow stops when button is released.
7. For use with machinery irrigation system, connect the Luer male connection to the three way tap of the machinery irrigation.

The irrigation set sterilized with ethylene oxide is for single use only. Do not resterilize. Do not re-use. Keep in clean and dry place, protect from moisture, direct rays of the sun and source of heat. Discard after use according with current legislation.

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