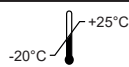
<b>REF</b>	Modello - Model - Modèle - Modell - Modelo	<b>MONOART® FFP2 NR PROTECTION</b>
<b>LOT</b>	Lotto - Lot - Lot - Chargennummer - Código de lote	
	Data di fabbricazione - Date of manufacture - Date de fabrication - Herstellungsdatum - Fecha de fabricación	
	Scadenza - Expiry - Date limite - Verfallsdatum - Vencimiento	
	Proteggere dall'umidità - Protect from moisture - Protéger de l'humidité - Vor Feuchtigkeit schützen - Proteger de la humedad	
	Limite di temperatura - Temperature limit - Limite de température - Temperaturgrenze - Límite de temperatura	
	No latex - Latex free - Sans latex - Latexfrei - No contiene látex	
	Non utilizzare se la confezione della semimaschera è danneggiata - Do not use if package of the half mask is damaged - Ne pas utiliser si l'emballage du demi-masque est endommagé - Nicht verwenden, wenn die Verpackung der Halbmaske beschädigt ist - No utilizar si el empaque de la media mascarilla está dañado	
	Cartone non ondulato - Non-corrugated cardboard - Carton non ondulé - Nichtwellpappe - Cartón no ondulado	
	Carta - Paper - Papier - Papier - Papel	
	Consultare le istruzioni per l'uso - Consult instructions for use - Consulter les instructions d'utilisation - Gebrauchsanweisung beachten - Consultar instrucciones de uso	
	Fabbricante - Manufacturer - Fabricant - Hersteller - Fabricante	

EN 149:2001 + A1:2009

CE 0068

REV.02/2021

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### EN

#### WARNINGS AND DISCLAIMERS

Before wearing the half mask visually verify that the device is in perfect condition, clean and intact. The half mask is not reusable and must be replaced after use or, at most, after a work shift. Modifications or alterations of the certified configuration aren't allowed.

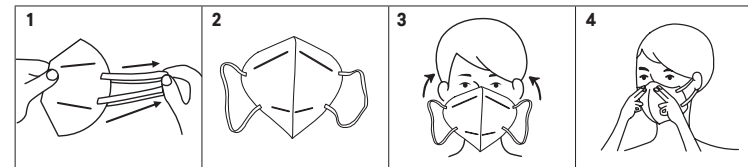
Check that the half mask adheres perfectly to the face. Covering nose, mouth and chin with your hands, inspire and exhale deeply. There should be no air flows. The presence of a beard, even if light, and a moustache that may protrude from the ends of the half mask, compromise the tightness and effectiveness of the device. Do not use half masks in oxygen-deficient environments (max 17% by volume), in the presence of harmful gases or vapors, in the presence of explosive atmospheres, in the presence of flames, when concentrations of contaminants are unknown or when substances are declared life-threatening. In case of dizziness, anxiety or other ailments, immediately leave the contaminated area. The increase in respiratory resistance, in very dusty environments, indicates that the half mask is clogged and must be immediately replaced. Immediately leave the operating area and replace the half mask in case of damage or detachment of the ear bands. If the half mask tends to lose tightness, exit the contaminated area. In case of atmospheres contaminated by viruses, bacteria, etc. absolutely avoid to fix the half mask by touching the outside with your fingers. The indications and uses not specifically contained in these instructions are not to be considered feasible. For more details / information contact the manufacturer.

The EU Type examination is conducted by / The notified body responsible for the ongoing conformity (module C2) is: MTIC Intercert S.r.l. (Notified Body No. 0068) via Leopardi, 14 Milano. The EU declaration of conformity is available on the following website: <https://monoart.euronda.it/dichiarazione-di-conformita/> Euronda S.P.A. declines all responsibility for damages caused by improper use of the PPE or in conflict with the content of these instructions.

#### How to wear Monoart® FFP2 NR Protection:

**Before wearing the half mask, wash hands with soap and water or with hydroalcoholic solution.**

1. Grip the half mask so that the rounded outer part is resting on the palm of the hand and the elastic bands remain under the hand.
2. Place the half mask under the chin with the nose wire at the top.
3. Place elastic bands behind the ears.
4. Shape the nose wire to the conformation of the nose. Check the sealing of the half mask by performing a sealing test before use inspiring and exhaling deeply. If air flows are perceived, reposition the half mask and repeat the breathing test.

**CHARACTERISTICS:** The FFP2 NR mask is a personal protective device, providing adequate protection of the respiratory tract in industrial environments where workers are exposed to dust and/or liquid particles, mists and fumes. It also has a filtering capacity >94% according to standard EN 149:2001+A1:2009, FPA 10 x TLV-TWA.

**STORAGE:** The half mask, in its original packaging, must be kept in a dry and clean environment at a temperature between -20 °C and + 25 °C with a relative humidity of <80%.

**EXPIRY DATE:** The expiry date of the product is 4 years from the date of production. The expiry date (month and year) is indicated with the appropriate pictogram on the packaging.

**TECHNICAL SPECIFICATIONS:** Filtering capacity and limits of use. The value of the filtering capacity is determined by the European Harmonized Standard EN 149:2001 +A1:2009 and the limit of use or Assigned Protection Factor (FPA) is established by European Standard EN 529:2005.

**DISPOSAL:** If the half mask has not been in contact with dangerous substances during use, it can be disposed of as solid urban waste, otherwise the specific legal requirements must be observed.

**MAINTENANCE, WASHING AND/OR DISINFESTATION:** NOT APPLICABLE. Disposable non reusable. The half mask must be disposed of at the end of use.

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