DENOMINAZIONE / PRODUCT
NAME / NOM DU PRODUIT /
PRODUKTBEZEICHNUNG /
DENOMINACIÓN / NOME DO
PRODUTO / NAZWA PRODUKTU

PROPRIETA' / PROPERTIES /
PROPRIÉTÉ / EIGENSCHAFTEN /
PROPIEDAD / PROPRIEDADE /
NIERUCHOMOŚĆ

CAMPI DI APPLICAZIONE /
APPLICATION AREAS /
DOMAINES D'APPLICATION /
ANWENDUNGSBEREICHE /
CAMPOS DE APLICACIÓN /
CAMPOS DE APLICAÇÃO /
OBSZARY ZASTOSOWAŃ

CARATTERISTICHE CHIMICO-
FISICHE / CHEMICAL AND
PHYSICAL CHARACTERISTICS /
CHARACTÉRISTIQUE PHYSIQUES
AND CHIMIQUES /

EUROCLEAN 120

DETERGENTE ENZIMATICO PER IL LAVAGGIO MECCANICO DI DISPOSITIVI MEDICI
ENZYME CLEANER FOR THE MECHANICAL CLEANING OF MEDICAL DEVICES
DÉTERGENT ENZYMATIQUE POUR LELAVAGE MÉCANIQUE DE DISPOSITIFS MÉDICAUX
ENZYME REINIGER FÜR DIE MECHANISCHES REINIGUNG VON MEDIZINISCHES GERÄTEN
DETERGENTE ENZIMÁTICO PARA EL LAVADO MECÁNICO DE DISPOSITIVOS MÉDICOS
DETERGENTE ENZIMÁTICO PARA A LAVAGEM MECÂNICA DOS DISPOSITIVOS MÉDICOS
CZYSZCZARKA ENZYMOVA DO MECHANICZNEGO CZYSZCZENIA WYROBÓW MEDYCZNYCH

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Enzymatic detergent for the mechanical washing of medical equipment. Detergent with enzyme action specifically for the preliminary removal of blood and organic contaminants from medical equipment to be disinfected or sterilised. It is recommended for mechanical and ultrasonic washing.

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Compatible with medical grade stainless steel, aluminium, plastic and rubber. Can be used for washing laboratory glassware too.

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pH sol. 0,1%

8,0 ± 0,5

Enzimi (proteasi, amilasi) / Enzymes (protease, amylase) /
Enzymes (protéase, amylase) / Enzyme (protease, amilase) /
Enzimas (proteasa y amilasa) / Enzimas (protease, amilase) /
Enzymy (proteazy, amylazy)

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Use within a temperature range of 40 and 65°C and at a concentration between 2 and 6 ml per liter of water, depending on the hardness of the water and the method of use.

Wash for 5 to 15 minutes, taking into account the type of machine, instrument and organic load.

PROFESSIONAL USE

Validity and storage

Validity: 2 years, stored in proper conditions.

Storage temperature: 5-30°C

Warnings

The concentrated product is irritating. Follow the advice in the material safety data sheet and on the label.

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