**DENOMINAZIONE / PRODUCT NAME / NOM DU PRODUIT / PRODUKTBEZEICHNUNG / DENOMINACIÓN / NOME DO PRODUTO / NAZWA PRODUKTU**

## EUROBRIGHT 360

**LIQUIDO DI RISCIAQUO LUBRIFICANTE PER IL LAVAGGIO DI STRUMENTARIO CHIRURGICO E DISPOSITIVI MEDICI  
LUBRICATING RINSE LIQUID FOR WASHING MACHINE OF SURGICAL INSTRUMENTS AND MEDICAL DEVICES  
LIQUIDE LUBRIFIANT POUR LE LAVAGE DES INSTRUMENTS ET DES DISPOSITIFS MÉDICAUX  
FLÜSSIG SPÜLUNGSMIERMITTEL FÜR WASHMACHINE VON CHIRURGISCHEN INSTRUMENTEN UND MEDIZINISCHES GERÄT  
DETERGENTE LUBRIFICANTE LIQUÍDO PARA LA ETAPA DE LAVADO DE LOS PRODUCTOS SANITARIOS  
DETERGENTE LUBRIFICANTE PARA FIM DE LAVAGEM DE INSTRUMENTOS E APARELHOS MÉDICOS  
PŁYN SMAROWY DO PŁUKANIA DO PRALEK NARZĘDZI CHIRURGICZNYCH I URZĄDZENIA MEDYCZNE**

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Detergent for use after washing. Concentrated detergent for rinsing the mechanical washing systems used for medical equipment, also set up for thermal disinfection. Continued use of EUROBRIGHT 360 facilitates drying and lubrication of the instrument, and makes the surfaces in medical grade stainless steel look brand new, protecting them against the effects of oxidation. Concentrated detergent for the rinsing-step in the mechanical washing systems of medical devices, washers, rotary-washing and tunnel-type machines and mechanical transport trolleys, also prepared with thermal disinfection. Free from silicones, neutral pH, with a drying additive. No-foaming; it does not leave traces on the treated surfaces. Maximum compatibility with polysulfones.

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1-2 ml/litre of water, which may be added to the final washing-water using automatic systems. Operating temperature 60 - 90°C. Preferably use demineralised water.  
PROFESSIONAL USE.

*Warnings*  
For automatic dosing via dosing pumps follow the indications given by the manufacturer of the washing machine by adjusting the dosage according to the water quality and the operating temperature. Keep out of children. Do not mix with other products.  
Before use read the label and consult the material safety data sheet.

*Validity and storage*  
Validity: 3 years, stored in proper conditions.  
Storage temperature: 5-30°C

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