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## SURGICAL ASPIRATOR

### MEDICAL DEVICE. NON-TOXIC AND NON PYROGENIC. LATEX AND PHTHALATES FREE.

If tubing is present, it contains phthalates. Use only if the package is not damaged or tampered with. Do not use the device if the caps are missing or detached.

Keep in a clean and dry place, away from moisture, direct sunlight and heat sources. The device is for single use. Do not reuse. Reuse can cause a loss of device functionality and a possible risk of patient infection. Use is only allowed to qualified personnel.

#### Instruction for use:

- 1) Verify the intended use of the disposal;
- 2) Open the package immediately before the use;
- 3) check the connections before proceeding with use, monitor the patient's condition during and after use, replace within 24 hours. Consider increasing the frequency of replacement if you have aspirated blood or vomit.

Should leaks occur during administration, replace the entire system.

The device must be thrown into a biohazard container. Dispose of according to national / international standards and hospital protocols.

Notice: In the event of an accident that has occurred in relation to the device, report the event to the manufacturer and the Competent Authority.

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