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Key
1 - One-way internal irrigation
1/2Y - Two-ways internal and external irrigation using the Y-connector provided
T - Tap to connect a second irrigation line

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270626 - 1 - Tubing extension for mechanical irrigation

270634 - 1/2Y - BA Intl Optima Surge OS 500, Optima Surge OS 600L, Saeshin Traus SIP 10, X Cube, De Giorgi Traus SIP 10

270635 - 1 - Bien Air Chiropro 980, BioSAFin Easy Surgery / MyTUTOR, EMS Piezon, Master Surgery*, Esacrom Surgysonic I / Surgysonic II / Surgysonic Moto

270636 - 1 - Nouvag TCM3000 / MD8000 / MD7000 / MD10 / MD10S / MD11 / MD20 / MD30

270637 - 1 - KaVo Intrasurg 300/500 / Intra surg 300 PLUS / Intrasurg 1000 Intrasurg / 1000 Air

270638 - 1 - De Giorgi Steril Intraplant / Intrasurgery 2.0 / Intramatic 2 / Intramatic Plus, Friatec AG

270639 - 1 - Suni Satelec 2000 / 3000, Suni Max Surgi set basic, Surgi set pro 500, Dental surgery unit, Dental Unit DSC Electronic, Sky unit, Bredent, Blue Rocket, BR3000, BR4000

270640 - 1/2Y - BA INTERNATIONAL: Optima Surge OS 600L

Euronda®

Alle

Infusion sets for pump and accessories (connectors, extension lines)

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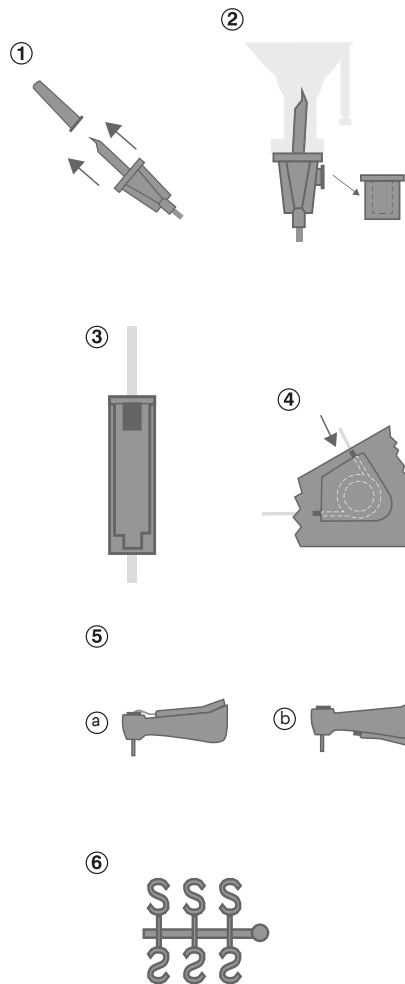
Irrigation set for peristaltic pump for the distribution of liquid coolants during oral surgery. Non-toxic. Apyrogenic. Free from latex and phthalates.

Instructions for use:

1. Remove the spike cap.
2. Insert the spike into the bag and connect the irrigation system using the Luer lock. The spike has an air filter with an open-close system, which allows you to regulate the flow rate.
3. Where necessary, use roller to adjust flow rate.
4. Insert the tube into the peristaltic pump, paying close attention to the flow direction and using the special connectors at the ends of the silicone tube.
5. If equipped with a tap, connect the end extension and adjust the direction of flow.
6. The irrigation systems offer the following options:
 - a. One-way internal irrigation
 - b. One-way external irrigation
 - c. Two-way internal and external irrigation, using the Y-connector provided.
7. Fix the irrigation tube to the handpiece tube using the clips provided.

Irrigation sets are sterilized with ethylene oxide and are single-use only. Do not resterilize. Do not re-use. Keep in a clean, dry place, protecting it from moisture, direct sun rays and heat sources. After use dispose in accordance with current legislation.

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