Sanifizer

Wipes

V. 04/2026

IT - Italiano	2
EN - English	3
FR - Français	4
ES - Español	5
DE - Deutsch	6
PT - Português	7
PL - Polski	8
CS - Čeština	9
DA - Dansk	10
ET - Eesti keel	11
SV - Svenska	12
BG - български	13
EL - Ελληνικά	14
HR - Hrvatski	15
HU - Magyar	16
LT - Lietuvių	17
FI - Suomalainen	18
SK - Slovenský	19
RO - Română	20
LV - Latviski	21
SL - Slovenščina	22
NL - Nederlands	23

REF 246006	EURONDA S.P.A. Via dell'artigianato 7, 36030 Montecchio Precalcino (VI), Italy	MADE IN ITALY	
# ADMR204.100	ADRANOX srl Via Imre Nagy 46, 46100 Mantova Italy	MD	CE 0051

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techinfo@euronda.com

Fields of use

Sanifizer Wipes is indicated for the following fields of use: Disinfection and simultaneous cleaning of all non-invasive medical devices surfaces such as: scialytics, operating beds, trolleys, monitors and touch screens, medical equipments, work plans, incubators and thermal cradles in general, hemodialysis equipments, surfaces in general of hospital wards.

Contact time and use

Intermediate level disinfection (bactericidal, yeasticidal, mycobactericidal and virucidal activity)	Ready to use	5 - 30 min
--	--------------	------------

Ready To Use. Sanifizer Wipes are pre-soaked and ready-to-use wipes, in flow pack packaging. Open the pop-up cap and take out a wipe. Apply and use the product directly on the surface to be treated. Do not rinse. Close the package carefully after application. Since Sanifizer Wipes do not need to be rinsed, the contact times indicated in the relevant table will potentially always be respected.

Antimicrobial activity

The biocidal activity tests of Sanifizer Wipes were performed according to the current standards published by the European Committee for Standardization (CEN TC 216) by Test Centers Certified as operating according to GLP standards (Good Laboratory Practices) and ISO 17025 certified.

Sanifizer Wipes has a biocidal activity as follow:

Effectiveness	EN Standards	Times
Bactericide (S.Aureus, P.Aeruginosa, E.Hirae)	EN 13727* EN 16615* (with mechanical action)	5 min
Yeasticide (Candida Albicans, Candida Auris)	EN 13624* EN 16615* (with mechanical action)	15 min
Mycobactericide (M.Terrae, M.Avium)	EN 14348	15 min
Virucidal (Norovirus, Adenovirus, HIV, HBV, HCV, Vaccinia Virus including Coronavirus like SARS CoV- 2 virus, Rotavirus)	EN 14476 EN 16777	15 min - 30 min (Adenovirus Norovirus)

*Tests performed under dirty conditions

Materials compatibility

Sanifizer Wipes, if used according to the indications for use, is compatible with: stainless steel, aluminum, polypropylenes (PPE), PVC, polystyrene, teflon (PTFE), polycarbonates. The product is not compatible with Copper and Zinc. In any case, for confirmation, and in case of materials not present in this list, you can contact the manufacturer.

Composition

100 g of ready-to-use solution contains: Didecyldimethylammonium chloride, chlorhexidine digluconate, alcohol, non-ionic surfactant and purified water qs to 100.

Storage information

Store tightly closed. Storage temperature: between 5°C and 35°C. Keep out of reach of children. Stability: 36 months (product not used) if stored according to the indications provided. Once opened for the first time, the product, in unopened packaging, is stable for 12 months as long as it is within the expiry date indicated on the label.

Warnings

Sanifizer Wipes is a ready-to-use product for professional use only. When using, follow the warnings on the label and in the safety data sheet. Dispose of used wipes and empty flow packs according to local regulations. Report any serious accident involving the user and/or patient that has occurred in connection with the use of the device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

For any questions or to request the instructions for use in paper form contact: techinfo@euronda.com

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