DENOMINAZIONE / PRODUCT NAME / NOM DU PRODUIT / PRODUKTBEZEICHNUNG / DENOMINACIÓN / NOME DO PRODUTO

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Always follow the doctor's prescription.

Do not use if the bag is damaged and do not break the bag. In case of bag content contact with skin or eyes, rinse with plenty of water. The bag content stains and is harmful if swallowed. If the bag content is ingested, consult a doctor immediately. The content is noxious. For external use only, do not use on open wounds. Irritation or sensitization may be caused by individual intolerance to cellulose and nonwoven.

In case of cardiovascular and circulatory diseases, diabetes and / or skin insensitivity (Raynaud's phenomenon), use the device only after consulting the doctor. Consult the doctor before using on new born, children, aged people and pregnant women. Keep out of children's reach. Do not freeze the unit. Absolute contraindications: Raynaud's phenomenon, cold allergy, cryoglobulinemia, and paroxysmal cold hemoglobinuria. Relative contraindications: arthritic conditions, pheochromocytoma, anesthetic skin, or a patient who is unresponsive due to cardiovascular disease.

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